

Acrylic Primer

Intended Purpose

Acrylic bonding agent, for use by a dental professional

Other Materials / Equipment Needed

1. Disposable brushes

Instructions for Use

1. Paint Acrylic Primer liquid over original acrylic surface to which new acrylic will be added.
2. Allow Acrylic Primer to dry completely.
3. Add new acrylic.

Note: Jet™ Adjusters Thinner can be used to dilute Acrylic Primer.

Handling Precautions

Contains methyl methacrylate

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surface. – No smoking.

Causes skin irritation. May cause an allergic skin reaction.

May cause respiratory irritation.

Wear protective gloves/protective clothing/eye protection/face protection.

Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Disposal

Dispose of contents and container in accordance with all local, regional, national and international regulations.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark® Europe. 11 rue E. Zola
38100 Grenoble. France



Lang Dental Mfg. Co., Inc.
175 Messner Dr.
Wheeling, IL 60090 USA
www.LangDental.com



Made in USA
Ph +1 847-215-6622
Fax +1 847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2020 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. JET is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 16 R. 8 2020-01